


Prealbumin / Transthyretin

Clone: BSB-125

Mouse Monoclonal

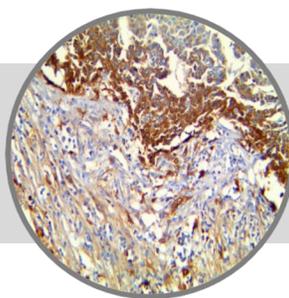

Bio SB
BIOSCIENCE FOR THE WORLD

www.biosb.com

Inset: IHC of Prealbumin / Transthyretin on a FFPE Hepatocellular Carcinoma Tissue

Intended Use

For In Vitro Diagnostic Use.

This antibody is intended for use in Immunohistochemical applications on formalin-fixed paraffin-embedded tissues (FFPE), frozen tissue sections and cell preparations. Interpretation of results should be performed by a qualified medical professional.

Immunogen

Synthetic peptide corresponding to the C-terminus of the human Prealbumin protein.

Summary and Explanation

TTR (TTR, transports thyroxine and retinol, or TBPA) encodes one of the three prealbumins including alpha-1-antitrypsin, transthyretin and orosomucoid. Transthyretin is a carrier protein; it transports thyroid hormones in the plasma and cerebrospinal fluid, and also transports retinol (vitamin A) in the plasma. The liver secretes Transthyretin into the blood, and the choroid plexus secretes TTR into the cerebrospinal fluid.

TTR mutations are associated with amyloid deposition, predominantly affecting peripheral nerves or the heart. TTR misfolding and aggregation is known to be associated with the Amyloid Diseases, Senile Systemic Amyloidosis (SSA), Familial Amyloid Polyneuropathy (FAP), and Familial Amyloid Cardiomyopathy (FAC). Transthyretin amyloidosis is a slowly progressive condition characterized by the buildup of abnormal deposits amyloid (Amyloidosis) in the body's organs and tissues.

It has been reported that TTR can be used as an immunohistochemical marker for choroid plexus papillomas, as well as carcinomas. The *TTR* gene has been found to be suppressed in Hepatic Carcinoma, where the *TTR* gene was found to be defective in its gene structure, which may have a relevance in its pathogenesis.

Antibody Type	Mouse Monoclonal	Clone	BSB-125
Isotype	IgG2b/K	Reactivity	Paraffin, Frozen
Localization	Cytoplasmic, Membranous	Control	Liver, Pancreas, Kidney, Leydig Cells, Hepatocellular Carcinoma, Seminoma
Species Reactivity		Human, Mouse, Rat	

Presentation

Prealbumin is a mouse monoclonal antibody derived from cell culture supernatant that is concentrated, dialyzed, filter sterilized and diluted in buffer pH 7.5, containing BSA and sodium azide as a preservative.

Catalog No.	Antibody Type	Dilution	Volume/Qty
BSB 3616	Tinto Prediluted	Ready-to-Use	3.0 mL
BSB 3617	Tinto Prediluted	Ready-to-Use	7.0 mL
BSB 3618	Tinto Prediluted	Ready-to-Use	15.0 mL
BSB 3619	Concentrated	1:50 - 1:200	0.1 mL
BSB 3620	Concentrated	1:50 - 1:200	0.5 mL
BSB 3621	Concentrated	1:50 - 1:200	1.0 mL

Control Slides Available

Catalog No.	Quantity
BSB 3622	5 slides

Precautions

1. For professional users only. Results should be interpreted by a qualified medical professional.
2. This product contains <0.1% sodium azide (NaN₃) as a preservative. Ensure proper handling procedures are used with this reagent.
3. Always wear personal protective equipment such as laboratory coat, goggles and gloves when handling reagents.
4. Dispose of unused solution with copious amount of water.
5. Do not ingest reagent. If reagent is ingested, seek medical advice immediately.
6. Avoid contact with eyes. If contact occurs, flush with large quantities of water.
7. Follow safety precautions of the heating device used for epitope retrieval (TintoRetriever Pressure Cooker or similar).
8. For additional safety information refer to Safety Data Sheet for this product.
9. For complete recommendations for handling biological specimens, please refer to the CDC document, "Guidelines for Safe Work Practices in Human and Animal Medical Diagnostic Laboratories" (see References in this document).

Storage Store at 2-8°C (Control Slides: Store at 20-25°C)

Stability

This product is stable up to the expiration date on the product label. Do not use after expiration date listed on package label. Temperature fluctuations should be avoided. Store appropriately when not in use, and avoid prolonged exposure to room temperature conditions.

Specimen Preparation

Paraffin sections: The antibody can be used on formalin-fixed paraffin-embedded (FFPE) tissue sections. Ensure tissue undergoes appropriate fixation for best results. Pre-treatment of tissues with heat-induced epitope retrieval (HIER) is recommended using Bio SB ImmunoDNA Retriever with Citrate (BSB 0020-BSB 0023), ImmunoDNA Retriever with EDTA (BSB 0030-BSB 0033) or ImmunoDNA Digestor (BSB 0108-0112). See reverse side for complete protocol. Tissue should remain hydrated via use of Bio SB Immuno/DNA Washer solutions (BSB 0029 & BSB 0042).

Frozen sections and cell preparations: The antibody can be used for labeling acetone-fixed frozen sections and acetone-fixed cell preparations.

Staining Procedure

1. Cut and mount 3-5 micron formalin-fixed paraffin-embedded tissues on positively charged slides such as Bio SB Hydrophilic Plus Slides (BSB 7028).
2. Air dry for 2 hours at 58° C.
3. Deparaffinize, dehydrate and rehydrate tissues.
4. Subject tissues to heat induced epitope retrieval (HIER) using a suitable retrieval solution such as ImmunoDNA Retriever with Citrate (BSB 0020-BSB 0023) or EDTA (BSB 0030-BSB 0033).
5. Any of three heating methods may be used:

a. TintoRetriever Pressure Cooker or Equivalent

Place tissues/slides in a staining dish or coplin jar containing the ImmunoDNA Retriever with Citrate or EDTA, and place on trivet in the pressure cooker. Add 1-2 inches of distilled water to the pressure cooker and turn heat to high. Incubate for 15 minutes. Open and immediately transfer slides to room temperature.

b. TintoRetriever PT Module or Water Bath Method

Place tissues/slides in a pre-warmed staining dish or coplin jar containing the ImmunoDNA Retriever with Citrate or EDTA at 95°-99° C. Incubate for 30-60 minutes.

c. Conventional Steamer Method

Place tissues/slides in a pre-warmed staining dish or coplin jar containing the ImmunoDNA Retriever with Citrate or EDTA in a steamer, cover and steam for 30-60 minutes.

6. After heat treatment, transfer slides in ImmunoDNA Retriever with Citrate or EDTA to room temperature and let stand for 15-20 minutes.
7. For manual staining, perform antibody incubation at ambient temperature. For automated staining methods, perform antibody incubation according to instrument manufacturer's instructions.
8. Wash slides with ImmunoDNA washer or DI water.
9. Continue IHC staining protocol. Wash slides between each step with ImmunoDNA washer solution.

Abbreviated Immunohistochemical Protocol

Step	ImmunoDetector AP/HRP	PolyDetector AP/HRP	PolyDetector Plus HRP
Peroxidase/AP Blocker	5 min.	5 min.	5 min
Primary Antibody	30-60 min.	30-60 min.	30-60 min.
1st Step Detection	10 min.	30-45 min.	15 min.
2nd Step Detection	10 min.	Not Applicable	15 min.
Substrate-Chromogen	5-10 min.	5-10 min.	5-10 min.
Counterstain / Coverslip	Varies	Varies	Varies

Mounting Protocols

For detailed instructions using biodegradable permanent mounting media such as XyGreen PermaMounter (BSB 0169-0174) or organic solvent based resin such as PermaMounter (BSB 0094-0097), refer to PI0174 or PI0097.

Product Limitations

Due to inherent variability present in immunohistochemical procedures (including fixation time of tissues, dilution factor of antibody, retrieval method utilized and incubation time), optimal performance should be established through the use of positive and negative controls. Results should be interpreted by a qualified medical professional.

References

1. Razavi H, et al. "Benzoxazoles as transthyretin amyloid fibril inhibitors: synthesis, evaluation, and mechanism of action". *Angew. Chem. Int. Ed. Engl.* 2003; 42 (24): 2758-61.
2. Zeldenrust SR, Benson MD. "Familial and senile amyloidosis caused by transthyretin". In Ramirez-Alvarado M, Kelly JW, Dobson C. *Protein misfolding diseases: current and emerging principles and therapies.* New York: Wiley. 2010; pp. 795-815.
3. Westermark P, et al. "Fibril in senile systemic amyloidosis is derived from normal transthyretin". *Proc. Natl. Acad. Sci. U.S.A.* 1990; 87 (7): 2843-5.
4. Holmgren G, et al. "Clinical improvement and amyloid regression after liver transplantation in hereditary transthyretin amyloidosis". *Lancet.* 1993; 341 (8853): 1113-6.
5. TTR transthyretin (Homo sapiens) <https://www.ncbi.nlm.nih.gov/gene/7276>
6. Li X, Buxbaum JN. "Transthyretin and the brain re-visited: is neuronal synthesis of transthyretin protective in Alzheimer's disease?". *Mol Neurodegener.* 2011; 6 (1): 79.
7. Gu JR, et al. Transthyretin (prealbumin) gene in human primary hepatic cancer. 1991; europepmc.org/abstract/MED/1666289
8. U.S. Department of Health and Human Services: Centers for Disease Control and Prevention. *Guidelines for Safe Work Practices in Human and Animal Medical Diagnostic Laboratories.* Supplement / Vol. 61, January 6, 2012.

Symbol Key / Légende des symboles/Erläuterung der Symbole

	EMERGO EUROPE Prinsessegracht 20 2514 AP The Hague The Netherlands		Storage Temperature Limites de température Zulässiger Temperaturbereich		Manufacturer Fabricant Hersteller		Catalog Number Référence du catalogue Bestellnummer
	In Vitro Diagnostic Medical Device Dispositif médical de diagnostic in vitro In-Vitro-Diagnostikum		Read Instructions for Use Consulter les instructions d'utilisation Gebrauchsanweisung beachten		Expiration Date Utiliser jusque Verwendbar bis		Lot Number Code du lot Chargenbezeichnung

Bio SB
BIOSCIENCE FOR THE WORLD



69 Santa Felicia Dr., Santa Barbara, CA 93117, USA
Tel. (805) 692-2768 | Tel. (800) 561-1145 | Fax. (805) 692-2769
E-mail: sales@biosb.com | Website: www.biosb.com