


Follicular Dendritic Cell

Clone: CNA.42

Mouse Monoclonal

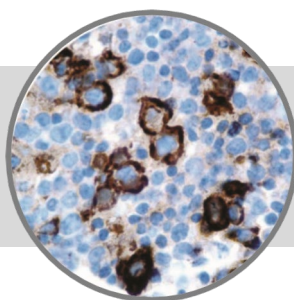

Bio SB
BIOSCIENCE FOR THE WORLD

www.biosb.com

Inset: IHC of Follicular Dendritic Cell on a FFPE Tonsil Tissue

Intended Use

For In Vitro Diagnostic Use.

This antibody is intended for use in Immunohistochemical applications on formalin-fixed paraffin-embedded tissues (FFPE), frozen tissue sections and cell preparations. Interpretation of results should be performed by a qualified medical professional.

Immunogen

Splenocytes from nude mice grafted with the human lymphoblastic CEM T-cell line.

Summary and Explanation

Follicular Dendritic Cells (FDC) are immune cells whose main function is to process antigen material and present it superficially to other cells of the immune system. Dendritic cells are present in small quantities in tissues that are in contact with the external environment, mainly the skin (where they are often called Langerhans cells) and the inner lining of the nose, lungs, stomach and intestines. They can also be found in an immature state in the blood. Once activated, they migrate to the lymphoid tissues where they interact with T-cells and B-cells to initiate and shape the immune response.

Anti-FDC is useful in the identification of follicular dendritic cell matrix found in normal lymph nodes and tonsillar tissue. This antibody has been found to label cells in approximately 60% of Anaplastic Large-Cell Lymphomas, and approximately 45% of T-cell Lymphomas. This antibody also labels Follicular Dendritic Cell Tumors. Several normal non-lymphoid tissues are labeled with anti-FDC: pancreatic islet cells, gastric chief cells, myelin sheaths, salivary glands, Leydig cells of the testis, and endothelial cells.

Antibody Type	Mouse Monoclonal	Clone	CNA.42
Isotype	IgM/K	Reactivity	Paraffin, Frozen
Localization	Cytoplasmic	Control	Tonsil, Lymph Node
Species Reactivity		Human, Cat, Cow, Mouse Rabbit, Rat, Sheep, Swine	

Precautions

1. For professional users only. Results should be interpreted by a qualified medical professional.
2. This product contains <0.1% sodium azide (NaN₃) as a preservative. Ensure proper handling procedures are used with this reagent.
3. Always wear personal protective equipment such as laboratory coat, goggles and gloves when handling reagents.
4. Dispose of unused solution with copious amount of water.
5. Do not ingest reagent. If reagent is ingested, seek medical advice immediately.
6. Avoid contact with eyes. If contact occurs, flush with large quantities of water.
7. Follow safety precautions of the heating device used for epitope retrieval (TintoRetriever Pressure Cooker or similar).
8. For additional safety information refer to Safety Data Sheet for this product.
9. For complete recommendations for handling biological specimens, please refer to the CDC document, "Guidelines for Safe Work Practices in Human and Animal Medical Diagnostic Laboratories" (see References in this document).

Presentation

Anti-FDC is a mouse monoclonal antibody derived from cell culture supernatant that is concentrated, dialyzed, filter sterilized and diluted in buffer pH 7.5, containing BSA and sodium azide as a preservative.

<i>Catalog No.</i>	<i>Antibody Type</i>	<i>Dilution</i>	<i>Volume/Qty</i>
BSB 5526	Tinto Prediluted	Ready-to-Use	3.0 mL
BSB 5527	Tinto Prediluted	Ready-to-Use	7.0 mL
BSB 5528	Tinto Prediluted	Ready-to-Use	15.0 mL
BSB 5529	Concentrated	1:25 - 1:100	0.1 mL
BSB 5530	Concentrated	1:25 - 1:100	0.5 mL
BSB 5531	Concentrated	1:25 - 1:100	1.0 mL

Control Slides Available

<i>Catalog No.</i>	<i>Quantity</i>
BSB 5532	5 slides

Storage Store at 2-8°C (Control Slides: Store at 20-25°C)

Stability

This product is stable up to the expiration date on the product label. Do not use after expiration date listed on package label. Temperature fluctuations should be avoided. Store appropriately when not in use, and avoid prolonged exposure to room temperature conditions.

Specimen Preparation

Paraffin sections: The antibody can be used on formalin-fixed paraffin-embedded (FFPE) tissue sections. Ensure tissue undergoes appropriate fixation for best results. Pre-treatment of tissues with heat-induced epitope retrieval (HIER) is recommended using Bio SB ImmunoDNA Retriever with Citrate (BSB 0020-BSB 0023), ImmunoDNA Retriever with EDTA (BSB 0030-BSB 0033) or ImmunoDNA Digestor (BSB 0108-0112). See reverse side for complete protocol. Tissue should remain hydrated via use of Bio SB Immuno/DNA Washer solutions (BSB 0029 & BSB 0042).

Frozen sections and cell preparations: The antibody can be used for labeling acetone-fixed frozen sections and acetone-fixed cell preparations.

Staining Procedure

- Cut and mount 3-5 micron formalin-fixed paraffin-embedded tissues on positively charged slides such as Bio SB Hydrophilic Plus Slides (BSB 7028).
- Air dry for 2 hours at 58° C.
- Deparaffinize, dehydrate and rehydrate tissues.
- Subject tissues to heat induced epitope retrieval (HIER) using a suitable retrieval solution such as ImmunoDNA Retriever with Citrate (BSB 0020-BSB 0023) or EDTA (BSB 0030-BSB 0033).
- Any of three heating methods may be used:
 - TintoRetriever Pressure Cooker or Equivalent**
Place tissues/slides in a staining dish or coplin jar containing the ImmunoDNA Retriever with Citrate or EDTA, and place on trivet in the pressure cooker. Add 1-2 inches of distilled water to the pressure cooker and turn heat to high. Incubate for 15 minutes. Open and immediately transfer slides to room temperature.
 - TintoRetriever PT Module or Water Bath Method**
Place tissues/slides in a pre-warmed staining dish or coplin jar containing the ImmunoDNA Retriever with Citrate or EDTA at 95°-99° C. Incubate for 30-60 minutes.
 - Conventional Steamer Method**
Place tissues/slides in a pre-warmed staining dish or coplin jar containing the ImmunoDNA Retriever with Citrate or EDTA in a steamer, cover and steam for 30-60 minutes.
- After heat treatment, transfer slides in ImmunoDNA Retriever with Citrate or EDTA to room temperature and let stand for 15-20 minutes.
- For manual staining, perform antibody incubation at ambient temperature. For automated staining methods, perform antibody incubation according to instrument manufacturer's instructions.
- Wash slides with ImmunoDNA washer or DI water.
- Continue IHC staining protocol. Wash slides between each step with ImmunoDNA washer solution.

Abbreviated Immunohistochemical Protocol

Step	ImmunoDetector AP/HRP	PolyDetector AP/HRP	PolyDetector Plus HRP
Peroxidase/AP Blocker	5 min.	5 min.	5 min
Primary Antibody	30-60 min.	30-60 min.	30-60 min.
1st Step Detection	10 min.	30-45 min.	15 min.
2nd Step Detection	10 min.	Not Applicable	15 min.
Substrate-Chromogen	5-10 min.	5-10 min.	5-10 min.
Counterstain / Coverslip	Varies	Varies	Varies

Performance Characteristics

Normal Tissues
Positive (+)
Secondary lymphoid follicles in reactive lymph nodes, tonsils, white pulp of spleen and Peyer's patches
In thymus, a subpopulation of large cortical thymocytes
In non-lymphoid organs the antibody labelled pancreatic islet cells, gastric chief cells, acini in salivary gland, Leydig cells of the testis, myelin sheaths, smooth muscles of the arterial wall, striated muscle fibers and endothelial cells
Abnormal Tissues
Positive (+)
malignant cells in 6/129 (5%) cases of B-cell lymphomas
malignant cells in 30/184 (16%) cases of T-cell lymphomas
malignant cells in 23/105 (22%) cases of Hodgkin's disease
malignant cells in 44/114 (39%) of diverse non-hematopoietic tumours such as different neurogenic tumours and carcinomas of gastrointestinal origin
In 11/11 cases of Epstein-Barr virus related inflammatory pseudotumour-like FDC tumours, the antibody labelled FDCs, scattered mononuclear cells, mast cells as well as and some epithelial and mesenchymal cells

Mounting Protocols

For detailed instructions using biodegradable permanent mounting media such as XyGreen PermaMunter (BSB 0169-0174) or organic solvent based resin such as PermaMunter (BSB 0094-0097), refer to PI0174 or PI0097.


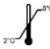






Product Limitations

Due to inherent variability present in immunohistochemical procedures (including fixation time of tissues, dilution factor of antibody, retrieval method utilized and incubation time), optimal performance should be established through the use of positive and negative controls. Results should be interpreted by a qualified medical professional.

References

- Raymond I, et al. Am J Pathol. 1997;Dec;151(6):1577-85
- Chan AC, et al. Histopathology. 2001;Jun;38(6):510-8
- Cheuk W, et al. Am J Surg Pathol. 2001;Jun;25(6):721-31
- Baur AS, et al. Histopathology. 1998;Jun;32(6):512-20
- Kunihiko M, et al. J Histochem & Cytothem. 2002;50(11):1475-85
- Pileri SA, et al. Histopathology. 2002;41,1-29
- U.S. Department of Health and Human Services: Centers for Disease Control and Prevention. Guidelines for Safe Work Practices in Human and Animal Medical Diagnostic Laboratories. Supplement / Vol. 61, January 6, 2012.

Symbol Key / Légende des symboles/Erläuterung der Symbole

	EMERGO EUROPE Prinsessegracht 20 2514 AP The Hague The Netherlands		Storage Temperature Limites de température Zulässiger Temperaturbereich		Manufacturer Fabricant Hersteller		Catalog Number Référence du catalogue Bestellnummer
	In Vitro Diagnostic Medical Device Dispositif médical de diagnostic in vitro In-Vitro-Diagnostikum		Read Instructions for Use Consulter les instructions d'utilisation Gebrauchsanweisung beachten		Expiration Date Utiliser jusque Verwendbar bis		Lot Number Code du lot Chargenbezeichnung

Bio SB
BIOSCIENCE FOR THE WORLD



69 Santa Felicia Dr., Santa Barbara, CA 93117, USA
Tel. (805) 692-2768 | Tel. (800) 561-1145 | Fax. (805) 692-2769
E-mail: sales@biosb.com | Website: www.biosb.com